

Anti-Human Insulin [EP125]

Catalog No.	Description
AN735-5M	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AN735-10M	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
NU735-UC	1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
NU735-5UC	0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AY735-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 160 tests
AY735-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 50 tests

Clone	Species	Ig Class
EP125	Rabbit	IgG

Intended Use

For In Vitro Diagnostic Use. This antibody is designed for the specific localization of Insulin in formalin-fixed, paraffin-embedded tissue sections. Evaluation must be performed by a qualified pathologist.

Summary and Explanation

Insulin is a hormone that regulates glucose homeostasis. It increases cell permeability to monosaccharides, amino acids and fatty acids, and it accelerates glycolysis, the pentose phosphate cycle, and glycogen synthesis in the liver. It is synthesized in the beta cell of the pancreas. One million to three million islets of Langerhans (pancreatic islets) form the endocrine part of the pancreas, which is primarily an exocrine gland. The endocrine portion accounts for only 2% of the total mass of the pancreas. Within the islets of Langerhans, beta cells constitute 65-80% of all the cells. The antibody labels both normal and neoplastic insulin-producing cells. It is useful in identifying insulinoma.

Storage and Handling

Store at 2-8°C. Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

Reagents Provided

Rabbit Monoclonal Antibody to Insulin is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	PANCREASas available with BiogenexFB-735N*& FG-735N*
Recommended Dilution for Concentrated Antibody	1:30-50 in HK941
Recommended Pretreatment (Manual/i6000)**	EZ-AR1 (HK521-XAK)
Recommended Pretreatment (Xmatrx)	EZ-AR1 Elegance (H031-YCD)
Antibody Incubation (Manual/i6000)	30-60 mins at RT
Antibody Incubation (Xmatrx)	30-60 mins at RT
Detection System for Manual, Xmatrx & i6000 systems***	Use BioGenex Two-Step OR One-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information

*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrx requires micro chamber slides.

**Pretreatment times will vary based on individual microwave power.

Category	Antibodies	Revision No.	J
Document No.	932-735N-EN	Release Date	11-Jan-2022

***For automation systems (Xmatrix-Elite, Xmatrix-Ultra & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Detection System	Two-Step HRP Kit	One-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKE (1000 Test)	QD630-XAKE (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKE (1000 Test)		
	QD420-YIKE (500 Test)	QD620-XAKE (500 Test)	QP900-9LE (500 Test)
	QD400-60KE (60 Test)		
Xmatrix - Automation	QD550-YCDE (200 Test)	QD610-YADE (200 Test)	N/A
i6000 - Automation	QD410-YAXE (200 Test)	QD610-YAXE (200 Test)	N/A
For more information, visit www.biogenex.com .			

Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or support@biogenex.com or your local distributor to report unusual staining.

Expected Results

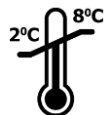





This antibody stains cytoplasm in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

Bibliography

- Dumonteil E, et al.: Diabetes Metab 1996, 22:164-173
- Alzira Martins Ferreira de Souza, et al.: Insulin or insulin-like studies on unicellular organisms: a review. Braz. arch. biol. technol. vol.47 no.6
- Dabbs DJ. Diagnostic Immunohistochemistry 2010; Churchill Livingstone
- Chang X, Jorgensen AM, et al.: "Solution structures of the R6 human insulin hexamer," Biochemistry 36 (31): 9409-22
- Roth J, et al.: Virchows Arch B Cell Pathol Incl Mol Pathol 1992,63:51-61
- Weidenheim KM, et al.: Am J Clin Pathol 1983, 79:14-24 100369

	Temperature Limitation	IVD	In Vitro Diagnostic Medical Device
	Use By Date	LOT	Batch Code
	Non-Sterile		Consult Instructions for Use
	Representative in the European Community		Manufacturer

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	J
Document No.	932-735N-EN	Release Date	11-Jan-2022