

## Anti-Human MSH2 [SP46]

Catalog No.	Description
AN743-5M	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AN743-10M	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
NU743-UC	1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
NU743-5UC	0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AY743-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 160 tests
AY743-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 50 tests

Clone	Species	Ig Class
SP46	Rabbit	IgG

### Intended Use

**For In Vitro Diagnostic Use.** This antibody is designed for the specific localization of MSH2 in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

### Summary and Explanation

MutS homologue 2 (MSH2) is a DNA mismatch repair protein that belongs to the MutS family. MSH2 forms two different heterodimers: MutS alpha (MSH2-MSH6) and MutS beta (MSH2-MSH3), which bind to DNA mismatches thereby initiating DNA repair. MSH2 is involved in DNA repair as a mismatch repair protein, and mutations of MSH2 are found in approximately 50% of inherited non polyposis colorectal carcinoma (HNPCC) (Lynch syndrome) cases. HNPCC is an autosomal, dominantly inherited disease associated with marked increase in cancer susceptibility. It is characterized by a familial predisposition to early onset colorectal carcinoma and extra-colonic cancers of the gastrointestinal, urological and female reproductive. Immunohistochemical analysis of MSH2 expression has been reported to be a practical and reliable method for the routine detection of the vast majority of MSI-H colorectal adenocarcinomas.

### Storage and Handling

**Store at 2-8°C.** Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

### Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

### Reagents Provided

Rabbit Monoclonal Antibody to MSH2 is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

### Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

### Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	COLON CAAs available with BiogenexFB-743N* & FG-743N*
Recommended Dilution for Concentrated Antibody	<b>1: 10-25 in HK941</b>
Recommended Pretreatment (Manual/i6000)**	EZ-AR2 (HK522-XAK) Twice
Recommended Pretreatment (Xmatrx)	EZ-AR2 Elegance (HX032-YCD)
Antibody Incubation (Manual/i6000)	30-60 Min at RT
Antibody Incubation (Xmatrx)	30-60 Min at 25°C
Detection System for Manual, Xmatrx & i6000 systems***	Use BioGenex Two-Step <b>OR</b> One-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information

\*FB: positive control micro chamberslides, FG: positive control microscopic slides. Xmatrx requires micro chamber slides.

\*\*Pretreatment times will vary based on individual microwave power.

Category	Antibodies	Revision No.	H
Document No.	932-743N-EN	Release Date	12-Nov-2021

\*\*\*For automation systems (Xmatrix-Elite, Xmatrix-Ultra & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Detection System	Two-Step HRP Kit	One-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKE (1000 Test)	QD630-XAKE (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKE (1000 Test)		
	QD420-YIKE (500 Test)	QD620-XAKE (500 Test)	QP900-9LE (500 Test)
	QD400-60KE (60 Test)		
Xmatrix - Automation	QD550-YCDE (200 Test)	QD610-YADE (200 Test)	N/A
i6000 - Automation	QD410-YAXE (200 Test)	QD610-YAXE (200 Test)	N/A
For more information, visit <a href="http://www.biogenex.com">www.biogenex.com</a> .			

### Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

### Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

### Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or [support@biogenex.com](mailto:support@biogenex.com) or your local distributor to report unusual staining.

### Expected Results

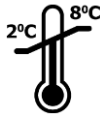





This antibody stains nucleus in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

### Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

### Bibliography

- Müller A, Fishel R. "Mismatch repair and the hereditary non-polyposis colorectal cancer syndrome (HNPCC)". *Cancer Invest.*20 (1): 102–9. .
- Guerrette S, et al.: *Mol Cell Biol*1998, 18:6616-6623
- Mellon I, et al.: "Transcription-coupled repair deficiency and mutations in human mismatch repair genes". *Science*272 (5261): 557–60.
- Edelbrock MA, et al.: "Structural, molecular and cellular functions of MSH2 and MSH6 during DNA mismatch repair, damage signaling and other noncanonical activities". *Mutat. Res*
- Warren JJ, et al.: "Structure of the human MutSalpα DNA lesion recognition complex". *Mol. Cell*26 (4): 579–92
- Wimmer K, et al.: *Hum Genet* 2008, 124:105-122
- Hardisson D, et al.: *Mod Pathol*2003, 16:1148-1158

	Temperature Limitation	<b>IVD</b>	In Vitro Diagnostic Medical Device
	Use By Date	<b>LOT</b>	Batch Code
	Non-Sterile		Consult Instructions for Use
	Representative in the European Community		Manufacturer

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	H
Document No.	932-743N-EN	Release Date	12-Nov-2021