

## Anti-MYOGENIN (Myf4) [MGN185]

| Catalog No. | Description   |
|-------------|---|
| AM987-5M    | 6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system                                   |
| AM987-10M   | 10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems |
| MU987-UC    | 1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system                                   |
| MU987-5UC   | 0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system                                 |
| AX987-YCD   | Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 160 tests   |
| AX987-50D   | Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 50 tests  |

| Clone  | Species | Ig Class    |
|--------|---------|-------------|
| MGN185 | Mouse   | IgG1, kappa |

### Intended Use

**For In Vitro Diagnostic Use.** This antibody is designed for the specific localization of myogenin protein antigen in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

### Summary and Explanation

Myogenin is a muscle-specific transcription factor that is responsible for the normal development of muscle fibers. Myf4 is part of the Myogenic regulatory factors family with MyoD, Myf5, and MRF4. This family of proteins contains a conserved DNA binding domain that binds to the E box DNA motif allowing regulation of myogenesis. Myogenin (Myf4) has also been known to transform non-muscle cells to muscle cells. Mice with homozygous null Myogenin (Myf4) gene tend to die soon after birth with severely underdeveloped muscles, therefore highlighting the importance of Myogenin (Myf4) in musculoskeletal development. Myogenin being muscle-specific can be used to identify muscle cells, or probes can be used to identify muscle cell precursors.

### Storage and Handling

**Store at 2-8°C.** Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

### Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

### Reagents Provided

Mouse Monoclonal Antibody to myogenin protein antigen is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

### Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

### Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

| Parameter  | BioGenex Recommendations  |
|--|---|
| Control Tissue   | Muscle tissue as available with Biogenex FB-987M* & FG-987M*  |
| Recommended Dilution for Concentrated Antibody         | <b>1:10-25 in HK941</b>   |
| Recommended Pretreatment (Manual/i6000)**              | EZ-AR1 (HK521-XAK)  |
| Recommended Pretreatment (Xmatrx)                      | EZ-AR1 Elegance (HX031-YCD)   |
| Antibody Incubation (Manual/i6000)                     | 30-60 min at RT   |
| Antibody Incubation (Xmatrx)                           | 30-60 min at 25°C   |
| Detection System for Manual, Xmatrx & i6000 systems*** | Use BioGenex Two-Step <b>OR</b> One-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information |

\*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrx requires micro chamber slides.

\*\*Pretreatment times will vary based on individual microwave power.

\*\*\*For automation systems (Xmatrx-Elite, Xmatrx-Ultra & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

|              |             |              |             |
|--------------|-------------|--------------|-------------|
| Category     | Antibodies  | Revision No. | F           |
| Document No. | 932-987M-EN | Release Date | 12-Nov-2021 |

| Detection System   | Two-Step HRP Kit       | One-Step HRP Kit       | Link and Label Kit     |
|--|------------------------|------------------------|------------------------|
| Manual   | QD440-XAKE (1000 Test) | QD630-XAKE (1000 Test) | QP300-XAKE (1000 Test) |
|  | QD430-XAKE (1000 Test) |                        |                        |
|  | QD420-YIKE (500 Test)  | QD620-XAKE (500 Test)  | QP900-9LE (500 Test)   |
|  | QD400-60KE (60 Test)   |                        |                        |
| Xmatrix - Automation   | QD490-YCDE (200 Test)  | QD610-YADE (200 Test)  | N/A                    |
| i6000 - Automation   | QD410-YAXE (200 Test)  | QD610-YAXE (200 Test)  | N/A                    |
| For more information, visit <a href="http://www.biogenex.com">www.biogenex.com</a> . |                        |                        |                        |

### Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

### Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

### Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or [support@biogenex.com](mailto:support@biogenex.com) or your local distributor to report unusual staining.

### Expected Results

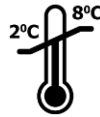







This antibody stains intercellular/nucleus in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

### Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

### Bibliography

- Hasty P., Bradley A., et al. Muscle deficiency and neonatal death in mice with a targeted mutation in the Myogenin gene. *Nature*. 1993 Aug 05; 364, pages 501-506.
- Grounds M., Garrett K., et al. Identification of skeletal muscle precursor cells in vivo by use of MyoD1 and Myogenin probes. *Springer-Verlag*. 1991 Jan 20. (1992) 267: 99.
- Perry R, Rudnick M. Molecular mechanisms regulating myogenic determination and differentiation. *Front Biosci*. 2000 Sep 1; 5:D750-67.
- Weintraub H., Davis R., et al. The myoD gene family: nodal point during specification of the muscle cell lineage. *Science*. 1991 Feb 15; 251(4995):761-6.
- Center for Disease Control. Decontamination of Laboratory Sink Drains to Remove Azide Salts. Center for Disease Control Manual Guide--Safety Management, No. CDC-22, Atlanta, Georgia. April 30, 1976.
- Kiernan JA. *Histological and Histochemical Methods: Theory and Practice*. New York: Pergamon Press 1981.
- Nadji M, Morales AR. Immunoperoxidase, part 1: the techniques and its pitfall. *Lab Med* 1983; 14:767-770.
- Omata M, Liew CT, Ashcavai M, Peters RI. Nonimmunologic binding of horseradish peroxidase to hepatitis B surface antigen. A possible source of error in immunohistochemistry. *Am J Clin Pathol*. May, 1980; 73(5):626-632.
- U.S. Congress. Clinical Laboratory Improvement Amendments of 1988: Final Rule, 57 FR 7163, February 28, 1992.
- National Institute for Occupational Safety and Health, (NIOSH), Rockville, MD. Explosive azide hazard, Publication No. 78-127, 1976.

|   |  |   |                                    |
|---|--|---|------------------------------------|
|  | Temperature Limitation                   |  | In Vitro Diagnostic Medical Device |
|  | Use By Date                              |  | Batch Code                         |
|  | Non-Sterile                              |  | Consult Instructions for Use       |
|  | Representative in the European Community |  | Manufacturer                       |

© 2020, BioGenex Laboratories. All rights reserved.

|                     |             |                     |             |
|---------------------|-------------|---------------------|-------------|
| <b>Category</b>     | Antibodies  | <b>Revision No.</b> | F           |
| <b>Document No.</b> | 932-987M-EN | <b>Release Date</b> | 12-Nov-2021 |