

## Anti-Serum Amyloid P [APCS/3240]

Catalog No.	Description
AMA68-5M	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AMA68-10M	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
MUA68-UC	1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
MUA68-5UC	0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AXA68-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 160 tests
AXA68-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 50 tests

Clone	Species	Ig Class
APCS/3240	Mouse	IgG2b

### Intended Use

**For In Vitro Diagnostic Use.** This antibody is designed for the specific localization of Serum Amyloid P in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

### Summary and Explanation

Serum Amyloid P (SAP), a member of pentraxin family, is a glycoprotein which has a characteristic pentameric organization. It is an acute phase protein, structurally related to C-reactive protein. SAP is made by hepatocytes and secreted into the blood. SAP is the single normal circulating protein that shows specific calcium-dependent binding to DNA and chromatin in physiological conditions. The avid binding of SAP displaces H1-type histones and thereby solubilizes native long chromatin, which is otherwise profoundly insoluble at the physiological ionic strength of extracellular fluids. The normal plasma protein serum amyloid P component (SAP) binds to fibrils in all types of amyloid deposits, and contributes to the pathogenesis of amyloidoses.

### Storage and Handling

**Store at 2-8°C.** Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

### Principles of the Procedure

Antigen detection by immunohistochemistry ([IHC](#)) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

### Reagents Provided

Mouse Monoclonal Antibody Serum Amyloid P is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

### Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

### Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	Liver tissue as available with Biogenex FB-A68M* & FG-A68M*
Recommended Dilution for Concentrated Antibody	<b>1:50-100 in HK941</b>
Recommended Pretreatment (Manual/i6000)**	EZ-AR2 (HK522-XAK)
Recommended Pretreatment (Xmatrx)	EZ-AR2 Elegance (HX032-YCD)
Antibody Incubation (Manual/i6000)	30-60 Min at RT
Antibody Incubation (Xmatrx)	30-60 Min at RT
Detection System for Manual, Xmatrx & i6000 systems***	Use BioGenex Two-Step <b>OR</b> One-Step Super Sensitive™ Polymer-HRP IHC Detection

Category	Antibodies	Revision No.	C
Document No.	932-A68M-EN	Release Date	11-Jan-2022

	System/DAB; see p. 2 for more information
--	---

\*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrix requires micro chamber slides.

\*\*Pretreatment times will vary based on individual microwave power.

\*\*\*For automation systems (Xmatrix-Elite, Xmatrix-Ultra & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Detection System	Two-Step HRP Kit	One-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKE (1000 Test)	QD630-XAKE (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKE (1000 Test)		
	QD420-YIKE (500 Test)	QD620-XAKE (500 Test)	QP900-9LE (500 Test)
	QD400-60KE (60 Test)		
Xmatrix - Automation	QD550-YCDE (200 Test)	QD610-YADE (200 Test)	N/A
i6000 - Automation	QD410-YAXE (200 Test)	QD610-YAXE (200 Test)	N/A
For more information, visit <a href="http://www.biogenex.com">www.biogenex.com</a> .			

### Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

### Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

### Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or [support@biogenex.com](mailto:support@biogenex.com) or your local distributor to report unusual staining.

### Expected Results

This antibody stains cytoplasm in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the

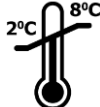





staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

### Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

### Bibliography

- Lewis JE, Olsen KD, Kurtin PJ, Kyle RA. Laryngeal Amyloidosis: A Clinicopathologic and Immunohistochemical Review. *Otolaryngology-Head and Neck Surgery*. 1992;106(4):372-377.
- Yang GC, Nieto R, Stachura I, Gallo GR. Ultrastructural immunohistochemical localization of polyclonal IgG, C3, and amyloid P component on the congo red-negative amyloid-like fibrils of fibrillary glomerulopathy. *Am J Pathol*. 1992;141(2):409-419.
- Koji Yasojima, Claudia Schwab, Edith G. McGeer, Patrick L. McGeer, Human neurons generate C-reactive protein and amyloid P: upregulation in Alzheimer's disease, *Brain Research*, Volume 887, Issue 1,2000,Pages 80-89.
- M B. Pepys, D. R. Booth, W. L. Hutchinson, J. R. Gallimore, I. M. Collins & E. Hohenester (1997) Amyloid P component. A critical review, *Amyloid*, 4:4, 274-295
- Emsley, J., White, H., O'Hara, B. *et al.* Structure of pentameric human serum amyloid P component. *Nature* **367**, 338-345 (1994).

	Temperature Limitation	<b>IVD</b>	In Vitro Diagnostic Medical Device
	Use By Date	<b>LOT</b>	Batch Code
	Non-Sterile		Consult Instructions for Use
	Representative in the European Community		Manufacturer

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	C
Document No.	932-A68M-EN	Release Date	11-Jan-2022