

Anti- YAP [G-6]

Catalog No.	Description
AMC50-5M	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AMC50-10M	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
AXC50-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite Staining System, 160 tests
AXC50-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite Staining System, 50 tests
AXC50-4M	Ready-to-Use Antibody in Barcode labeled vial for use on the NanoVIP® Staining System, 50 tests

Clone	Species	Ig Class
G-6	Mouse	IgG1

Intended Use

For In Vitro Diagnostic Use. This antibody is designed for the specific localization of YAP in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

Summary and Explanation

YAP (Yes-associated protein) also designated as YAP65, YAP1 is a 65 kDa 14-3-3-binding molecule that was originally recognized by virtue of its ability to bind to the SH3 domain of Yes. In addition to the SH3 binding motif, it contains a PDZ interaction motif, a coiled-coil domain, and WW domains. YAP encodes a nuclear effector of the Hippo signaling pathway which plays a pivotal role in regulating tissue growth and organ size. Deregulation of the Hippo pathway causes tumor formation and malignancy and hence, YAP may function as a potential target for cancer treatment. YAP is expressed at high levels in the lung, placenta, prostate, testis and ovary.

Storage and Handling

Store at 2-8°C. Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

Reagents Provided

Mouse Monoclonal Antibody YAP is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	Breast carcinoma tissue as available with Biogenex FB-C50M* & FG-C50M*
Recommended Pretreatment (Manual/i6000)**	EZ-AR2 (HK522-XAK)
Recommended Pretreatment (Xmatrx & NanoVIP)	EZ-AR2 Elegance (HX032-YCD & HX046-08XN)
Antibody Incubation (Manual/i6000)	30-60 Min at RT
Antibody Incubation (Xmatrx & NanoVIP)	30-60 Min at 25°C
Detection System for Manual, Xmatrx, NanoVIP & i6000 systems***	Use BioGenex Two-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information

*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrx & NanoVIP requires micro chamber slides.

**Pretreatment times will vary based on individual microwave power.

Category	Antibodies	Revision No.	A
Document No.	932-C50M-EN	Release Date	20-Apr-2022

***For automation systems (Xmatrx-Elite, NanoVIP & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Experimental results should be confirmed by a medically-established diagnostic product or procedure.

Detection System	Two-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKEN (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKEN (1000 Test)	
	QD420-YIKEN (500 Test)	QP900-9LE (500 Test)
	QD400-60KEN (60 Test)	
Xmatrx - Automation	QD550-YCDEN (200 Test)	N/A
NanoVIP- Automation	QD551-YCDEN (100 Test)	N/A
i6000 - Automation	QD410-YAXEN (200 Test)	N/A
For more information, visit www.biogenex.com .		

Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or support@biogenex.com or your local distributor to report unusual staining.

Expected Results

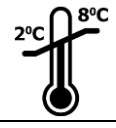





This antibody stains cytoplasm and nucleus in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user.

Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

Bibliography

1. Sudol, M., et al. 1995. Characterization of the mammalian YAP (Yes-associated protein) gene and its role in defining a novel protein module, the WW domain. *J. Biol. Chem.* 270: 14733-14741.
2. Basu, S., et al. 2003. Akt phosphorylates the Yes-associated protein, YAP, to induce interaction with 14-3-3 and attenuation of p73-mediated apoptosis. *Mol. Cell* 11: 11-23.
3. Danovi, S.A., et al. 2008. Yes-associated protein (YAP) is a critical mediator of c-Jun-dependent apoptosis. *Cell Death Differ.* 15: 217-219.

	Temperature Limitation	IVD	In Vitro Diagnostic Medical Device
	Use By Date	LOT	Batch Code
	Non-Sterile		Consult Instructions for Use
	Representative in the European Community		Manufacturer

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	A
Document No.	932-C50M-EN	Release Date	20-Apr-2022